- NL **Anti-allergieapparaat**  
FR **Appareil anti-allergies**  
DE **Anti-Allergiegerät**  
GB **Anti-allergy device**  
ES **Aparato antialérgico**  
PT **Dispositivo antialérgico**

Art. 45062

CE 0197

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E-Mail: [info@medisana.de](mailto:info@medisana.de)

Internet: [www.medisana.de](http://www.medisana.de)

Andon Health Co., Ltd.  
No. 3 Jinping Street,  
YaAn Road, Nankai District,  
Tianjin, 300190,  
China

Lotus Global Co.,Ltd.  
1 Four Seasons Terrace,  
West Drayton, Middlesex,  
London, UB7 9GG,  
United Kingdom

## Introduction

**Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use.**

Introduction Thank you for choosing the **MEDISANA Medinose Pro** Anti-allergy device. This product is a Class IIa medical device that uses technology proven in clinical trials for the treatment of Allergic Rhinitis including some symptoms of Hayfever (seasonal Allergic Rhinitis).

Millions of people suffer from Allergic Rhinitis and/or Hayfever. The suffering is often a lot worse in the summer months due to the presence of pollen and sufferers can become desperate to relieve the symptoms of their allergies.

The **MEDISANA Medinose Pro** is an innovative and effective treatment for the relief of irritating symptoms associated with Allergic Rhinitis specifically sneezing, running nose and running eyes. It is safe, quick and easy to use and some people may feel an improvement after just a few treatments.

These symptoms above are caused by the body's reaction to allergens. When the body comes into contact with allergens (e.g. pollen, dust, pet hair), it produces increased amounts of antibodies which cause certain cells to release histamine. It is histamine that triggers the symptoms of Allergic Rhinitis and/or Hayfever.

The **MEDISANA Medinose Pro** works by using phototherapy of two different wavelengths. One wavelength promotes increased blood flow and circulation and the second wavelength (red light therapy) suppresses the cells that release histamine and reduces inflammation, thereby relieving the irritating symptoms of Allergic Rhinitis.

The **MEDISANA Medinose Pro** can be used alone or alongside your normal allergy medication. Before using the **MEDISANA Medinose Pro**, please read this user manual carefully.

## Safety Information !

1. Children under 16 must be supervised by an adult when using this product.
2. Keep the device out of the reach of children.
3. Not suitable for use with a pacemaker or other implanted medical devices.
4. Not suitable for persons with Nasal Polyposis, sensitive skin in the nose or with poor circulation.
5. Never look directly into the light beams emitted, even when using an optical instrument (for example - glasses).
6. Avoid using the device in the vicinity of flammable or anaesthetic gases.
7. To avoid cross contamination, do not share nose adapter. Please purchase additional nose adapter for other users.


## Using the Medinose Pro

1. Insert the two nose adapter **6** into the nose as far as it is comfortably possible.
2. Press the **ON/OFF** button **1** once to turn the unit on. The redlight in two wavelengths are activated.
3. One treatment cycle lasts for approximately three minutes. The unit will automatically turn off after the treatment cycle is over. If you need to turn the unit off during treatment, press the **ON/OFF** button **1**.

The **MEDISANA Medinose Pro** can be used three to four times per day, allowing five to six hours between each treatment.

**Note:** If you feel any discomfort or pain at any time during treatment, turn the unit off and stop using immediately. Consult your doctor for advice.


### How to use wire:

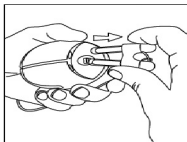1. Slide the nose adapter **6** out of the holder on the rear of the device.

2. Unwind the wire from the device and place around the neck.

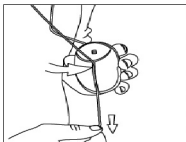3. Clip the wire into the rear slot to secure.


4. Adjust the position of the device on the wire.

## Battery Information

Open the battery compartment cover **3** by pulling the hook at the back of the unit and insert 2 x AAA batteries (included), observing correct polarity (as shown on the inside of the battery compartment).

When the battery power is low, the light on the **ON/OFF** button **1** will flash to indicate that the battery should be replaced.

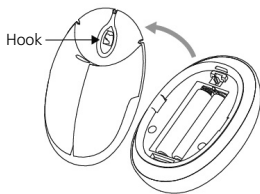- Remove the battery if the product is not in use for long periods of time.
- Do not mix old and new batteries or different types of batteries.
- **Warning:** If batteries leak and come into contact with the skin or eyes, wash immediately with copious amounts of water.
- Batteries should be handled under adult supervision.
- Dispose of batteries safely according to the battery manufacturer's instructions.

## Specification

Product Name:	<b>MEDISANA Medinose Pro</b> Anti-allergy device
Model Number:	AN-6019
Medisana Article Number:	45062
Technology of Treatment:	Phototherapy using 2 wavelength
Illuminate Position:	Nostrils
Wavelengths used:	652 nm and 940 nm (nanometers)
Beam Mode:	Continuous Mode
Operating Time:	Approximately 3 minutes
Operating Temperature/Humidity:	15°C ~ 40°C (59°F ~ 104°F) / 30% - 85% RH
Storage Temperature/Humidity:	-20°C ~ 60°C (-41°F ~ 140°F) / 30% - 85% RH
Battery Type:	2 x AAA batteries (included)
Battery Life:	Approximately 100 uses (~3 mins each use)
Product Dimension:	86 x 60 x 38 mm
Product Weight:	80 g (not include batteries)
Accessories:	Replacement nose adapter: Article Number 45067

### Electromagnetic compatibility:

The device complies with the EN 60601-1-2 standard for electromagnetic compatibility. Details of these measurements can be found at the end of this instruction manual.

## Maintenance and cleaning

1. Remove the battery if the product is not in use for long periods of time to prevent any damage that may be caused by battery deterioration or leakage.
2. This product is not waterproof. Never use aggressive cleaning agents or stiff brushes. To clean the unit, use a damp cloth to wipe the surface. Do not allow any liquid to enter the main unit.
3. Only use **MEDISANA** brand probes and cables with this device. Using nonapproved replacement parts may damage the unit and void the warranty.

## Explanation of symbols on unit

Read the instructions.

This symbol indicates that this product is a Type BF device.

Symbol for "Environment Protection" – Waste electrical products should not be disposed of with household waste. Please recycle where facilities exist. Check with your local authority or retailer for recycling advice.

Symbol for "Manufacturer".

This product complies with MDD93/42/EEC requirements.

Symbol for "European Representative".

Keep Dry.

Model Reference.

**In accordance with our policy of continual product improvement, we reserve the right to make technical and visual changes without notice.**

The current version of this instruction manual can be found under [www.medisana.com](http://www.medisana.com)

## Warranty and repair terms

Please contact your dealer or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is.

The following warranty terms apply:

1. The warranty period for **MEDISANA** products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be corrected free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
  - a. All damage which has arisen due to improper treatment, e.g. nonobservance of the user instructions.
  - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
  - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
  - d. Accessories which are subject to normal wear and tear (batteries etc.).
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

The service centre address is shown on the attached leaflet.

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**Table 1**

**For all ME EQUIPMENT and ME SYSTEMS**

**Guidance and manufacture's declaration - electromagnetic emissions**

The AN-6019 is intended for use in the electromagnetic environment specified below. The customer or the user of the AN-6019 should assure that it is used in such an environment.

<b>Emissions test</b>	<b>Compliance</b>	<b>Electromagnetic environment-guidance</b>
RF emissions CISPR 11	Group 1	The AN-6019 uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment
RF emissions CISPR 11 Harmonic emissions IEC 61000-3-2	Class B  Not applicable	The AN-6019 is suitable for use in all establishments other than domestic and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Voltage fluctuations/flicker emissions IEC 61000-3-3	Not applicable	

**Table 2**

**For all ME EQUIPMENT and ME SYSTEMS**

**Guidance and manufacturer's declaration - electromagnetic immunity**

The AN-6019 is intended for use in the electromagnetic environment specified below. The customer or the user of the AN-6019 should assure that it is used in such an environment.

<b>IMMUNITY test</b>	<b>IEC 60601test level</b>	<b>Compliance level</b>	<b>Electromagnetic environment-guidance</b>
Electrostatic discharge (ESD) IEC 61000-4-2	± 6 kV contact ± 8 kV air	± 6 kV contact ± 8 kV air	Floors should be wood, concrete or ceramic tile.If floors are covered with synthetic material, the relative humidity should be at least 30 %. Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	

NOTE:  $U_T$  is the a.c. mains voltage prior to application of the test level.

**Table 3**

**For ME EQUIPMENT and ME SYSTEMS that are not LIFE-SUPPORTING  
Guidance and manufacturer's declaration - electromagnetic immunity**

The AN-6019 is intended for use in the electromagnetic environment specified below. The customer or the user of the AN-6019 should assure that it is used in such an environment.

IMMUNITY test	IEC 60601 test level	Compliance level	Electromagnetic environment-guidance
Radiated RF	3 V/m 80 MHz	3 V/m	Portable and mobile RF communications equipment should be used no closer to any part of the AN-6019, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.
IEC 61000-4-3	to 2.5 GHz		<p><b>Recommended separation distance:</b></p> <p>80 MHz to 800 MHz</p> $d = 1.2\sqrt{P}$ <p>800 MHz to 2,5 GHz</p> $d = 2.3\sqrt{P}$ <p>Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey,<sup>a</sup> should be less than the compliance level in each frequency range.<sup>b</sup></p> <p> Interference may occur in the vicinity of equipment</p> <p>Marked with the following symbol:</p>

NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the the location in which the AN-6019 is used exceeds the applicable RF compliance level above, the AN-6019 should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the AN-6019.

b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than [V1] V/m.

**Table 4**  
**For ME EQUIPMENT and ME SYSTEMS that are not LIFE-SUPPORTING**  
**Recommended separation distances between**  
**portable and mobile RF communications equipment and the AN-6019**

The AN-6019 is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the AN-6019 can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the AN-6019 as recommended below, according to the maximum output power of the communications equipment

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter		
	$d = 1.2\sqrt{P}$ 150 kHz to 80 MHz	$d = 1.2\sqrt{P}$ 80 MHz to 800 MHz	$d = 2.3\sqrt{P}$ 800 MHz to 2.5 GHz
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance  $d$  in meters (m) can be determined using the equation applicable to the frequency of the transmitter, where  $P$  is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

*Distributed by:*

**MEDISANA AG**

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41468 NEUSS

GERMANY

E-Mail: [info@medisana.de](mailto:info@medisana.de)

Internet: [www.medisana.de](http://www.medisana.de)

Andon Health Co., Ltd.

No. 3 Jinping Street,

YaAn Road, Nankai District,

Tianjin, 300190, China

**EC REP** Lotus Global Co.,Ltd.

1 Four Seasons Terrace,

West Drayton, Middlesex,

London, UB7 9GG,

United Kingdom

Made in PRC

**CE**  
0197

This product is a Class IIa approved medical device.